

## **Facts**

## Challenge

Severe disfiguration and facial fractures of an accident victim led to a situation wherein two patient-specific facial implants had to be designed and developed to aid in his recovery.

#### Solution

An EOS M 290 printer with Ti6Al4V ELI alloy was used to manufacture two patient-specific implants that could enable the trauma patient to make a full recovery from his facial injuries.

## Results

- Fast: Print time of 1.5 Hrs ensures short lead times
- Economic: Total powder consumption of only 32 grams
- Beneficial: Shorter surgery procedure, faster healing process and great aesthetic result



CT image of the zygomatic and orbital bone implant after surgery (Source: CSIR-CSIO)

EOS & CSIR-CSIO delivers customised implants for facial trauma patient



# Metal 3D printing helps restore the patient's natural features and regain quality of life

## Short profile

The Central Scientific Instruments Organization (CSIO), a constituent of the Council of Scientific and Industrial Research (CSIR), is a premier national laboratory dedicated to the research, design and development of scientific and industrial instruments. It is a multidisciplinary and multi-dimensional apex industrial R&D organization in India to stimulate growth of the instrument industry while covering a wide range of applications.

## Further information

Central Scientific Instruments
Organization (CSIR-CSIO)
Sector-30C, Chandigarh160030, India
www.csio.res.in

Additive manufactured patient-specific implants successfully helped trauma victims recover from physical disfigurations in Chandigarh, India

Customized implants and surgical guides have been widely adopted for the reconstruction of cranio-maxillofacial defects due to better performance and fit over their generic counterparts. With precise adaptation to patient's anatomy at region of implantation, surgeons are able to reduce surgical time and improve cosmesis.

Application of 3D modelling helps to change the way surgeons plan and execute post trauma surgeries with patient specific implants. Additive manufacturing helps to eliminate the constraints of freeform shape and structures, makes it possible for the fabrication of implants that conform to the physical and mechanical requirements of the region of implantation. Moreover additive manufacturing, by Direct Metal Laser Sintering (DMLS), enables on demand manufacturing of affordable Ti6Al4V ELI custom implants within acceptable delivery lead-time.

## Challenge

Patient, 48 year old male, suffers severe cranio-maxillofacial injury due to trauma. Diagnosed with multiple left frontal facial fractures, malunion was observed when patient is back for post-surgery facial reconstruction 6 months later. CSIR-CSIO's senior scientist, Mr. Vijay Kumar Meena, was approached to design customize implants to help rectify the facial misalignment

with minimum damage to the malunited bones.


## Solution

Mr Meena and his team uses 3D Modelling software, to create a 3D model of patient's anatomy depicting the defect, analyze and design the custom implants required to camouflage & correct the misalignment. Two patient specific implants were designed to support the reconstruction

surgery. One zygomatic implant was designed to camouflage the zygomatic depression. One orbital floor implant was designed to realign and correct the patient's eye-level.

Customized implant designs were reviewed and endorsed by the surgeon. The surgeon commented that the free-form curvature of the implants fits perfectly well with patient's facial and orbital floor curvature.



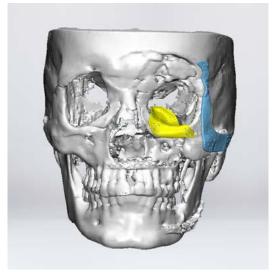

Scan of the patient before surgery (left) and designed implants over the model (right). Source: CSIR-CSIO

Auxein Medical Pvt Ltd, Sonipat, financially sponsored the manufacturing of the implants as part of their corporate social responsibility initiatives. EOS DMLS (Direct Metal Laser Sintering) technology was selected to manufacture the implants as high resolution and reliable quality of Ti6AI4V ELI implants was required. This technology enables custom manufacturing of complex structures with free-form curvatures in a fast and cost efficient manner. Thus the custom titanium (Ti6AI4V ELI) implants were fabricated with EOS M290 system.

After the reconstruction surgery, post-surgical follow up and radiological assessments have showed significant improvement in both hard and soft tissue profile.

The advantages of using patientspecific implants were subsequently documented as follows:

- Shorter surgery procedure
- Lower invasiveness of surgical treatment, lower risks of surgical and post-surgical complications
- Faster healing process, shorter post-surgical rehabilitation
- Lower risk of revision surgeries
- Reduced total treatment cost and duration
- Great aesthetic result for patient

The patient-specific implants

#### Results

manufactured using EOS DMLS technology was successfully used to correct the patient's eye-level and zygomatic imbalance. Speedy patient recovery was observed with no complications documented till date.

CSIR-CSIO has since established a center for the design and manufacture of customized affordable solutions for orthope-

dic and maxillofacial patients. With the acquisition of the EOS M290 industrial 3D printer, the facility is the pioneer in India that provides one stop solution to patients and orthopedic surgeons with custom specific implants. There has been an increase in demand for patient-specific implants in the maxillofacial regions, fractured acetabular cups and hip revision surgeries. This has enabled CSIR-CSIO to augment patient recovery by providing them with economic patient-specific implants in India with minimum lead times.

"Since the patient face was heavily disfigured by trauma, the patient-specific correction implants had several complex and freeform curves. The zygomatic implant and orbital floor implant had to be designed to prevent bending at the hands of surgeons in the operation theatre. Thanks to the additive manufacturing solution provided by EOS, the implant curvatures could be easily printed in a bio-compatible alloy that made the entire process seamless and successful."

Vijay Kumar Meena,
Senior Scientist,
Biomedical Instrumentation
Central Scientific Instruments
Organization (CSIR-CSIO),
Chandigarh, India
Email: vijaykumar@csio.res.in

EOS GmbH Electro Optical Systems Corporate Headquarters Robert-Stirling-Ring 1 D-82152 Krailling/Munich Germany Phone +49 89 893 36-0 Fax +49 89 893 36-285

Further EOS Offices

EOS France Phone +33 437 49 76 76

EOS Greater China Phone + 86 21 602307 00

EOS India Phone +91 44 39 64 80 00

EOS Italy Phone +39 02 33 40 16 59

EOS Japan Phone +81 4567 00250

EOS Korea Phone +82 2 63 30 58 00

EOS Nordic & Baltic Phone +46 31 760 46 40

EOS of North America Phone +1 248 306 01 43

EOS Singapore Phone +65 6430 04 63

EOS UK Phone +44 1926 67 51 10

www.eos.info • info@eos.info

Think the impossible. You can get it.

